

Saliva Instructions

ORAGENE DX COLLECTION



NOTE: For optimal results, complete saliva sample within **10 minutes**. For questions or concerns regarding your patient's saliva sample, please contact your Ambry representative.

Step 1




1 Open the new saliva collection kit

1 Remove the saliva collection tube

1 Write patient's name, date of birth, and collection date on the tube

Step 2

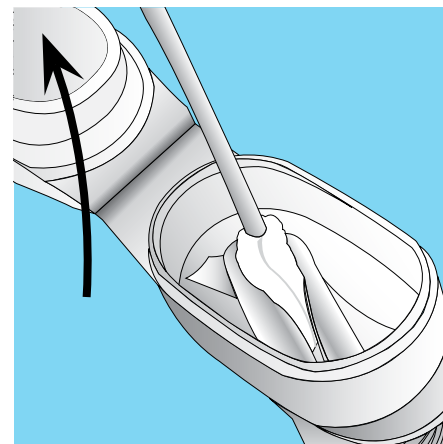

2 Do not puncture or remove the plastic film

2 Lid contains a clear solution


Step 3



3 Instruct patient to **NOT** eat, drink, smoke, or chew gum for **30 minutes** before giving their sample

PATIENT TIP: Before delivering saliva sample, relax and rub your cheeks gently for 30 seconds to help create saliva


Step 4

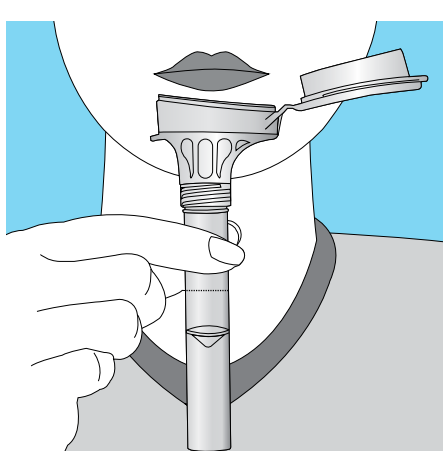

4 For optimal results, please complete saliva sample within 10 minutes


PATIENT TIP: Imagine candy, lemon, or a favorite food in your mouth

Step 5

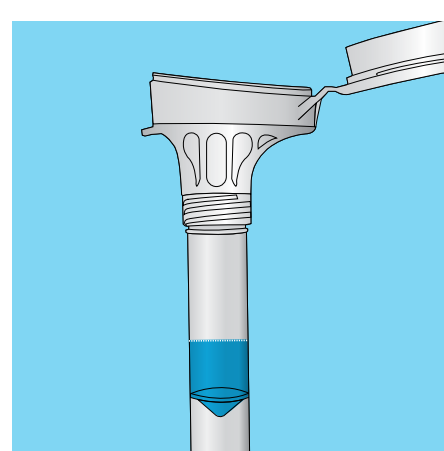


5 Spit until liquid saliva, **NOT** bubbles, reaches the fill line

Step 6

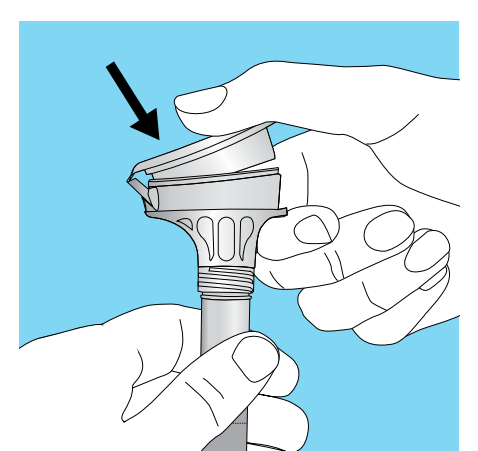


6 Once patient has reached the fill line, hold the tube upright and close the lid by firmly pushing down until you hear a loud click

Step 7

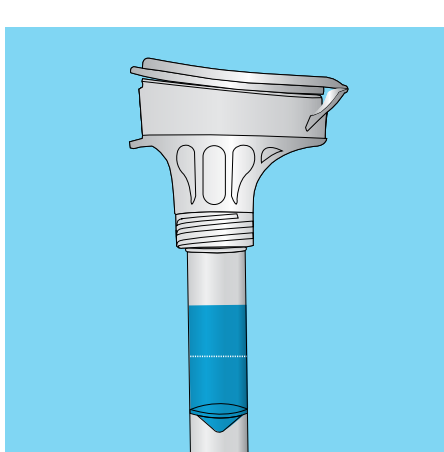


7 Once the clear fluid is released into the tube, the liquid will be higher than the actual fill line on the tube

Step 8

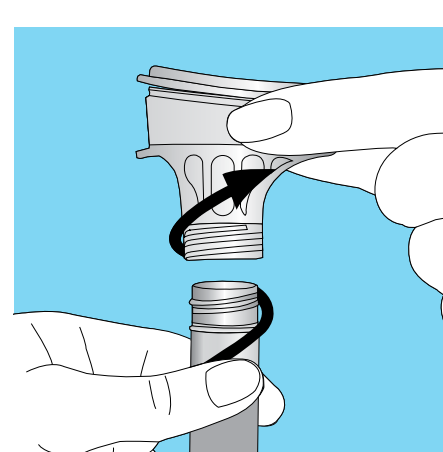


8 Remove funnel from the tube and discard

Step 9

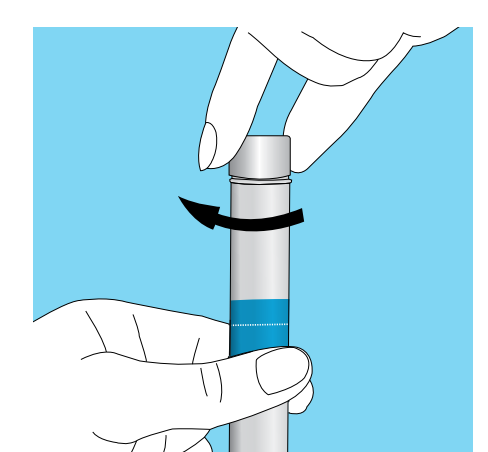

9 Use small cap to close the tube tightly

Step 10

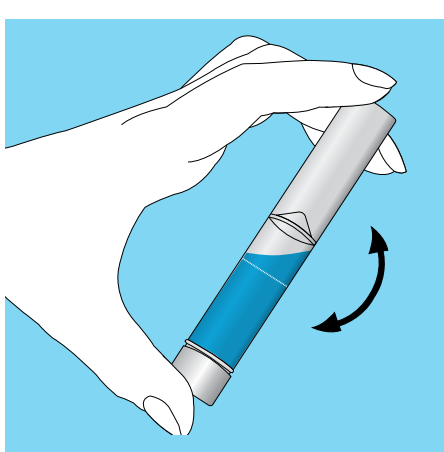

10 Invert the capped tube 5 times

10 Label tube with patient's name and date of birth and place back in plastic box

