

CancerNext-Expanded®

Product Summary

As Ambry's most comprehensive pan-cancer test option for hereditary cancer predisposition, CancerNext-*Expanded* includes genes associated with a wide range of hereditary cancers such as breast, ovarian, uterine, colorectal, gastric, pancreatic, prostate, melanoma, renal, central nervous system tumors, pheochromocytoma/paraganglioma, hematologic malignancy, and other rare cancer predisposition conditions.

Guidelines Recommend Genetic Testing For Hereditary Cancer

The NCCN Clinical Practice Guidelines in Oncology (NCCN Guidelines[®]) recommend that hereditary cancer testing be considered in appropriate individuals when it is likely to impact any of following:

- a patient's cancer risk management
- a patient's cancer treatment
- the medical management of a patient's at-risk family members^{1,2}

The NCCN Guidelines[®] also explain that because an individual's personal/family history may be explained by more than one gene or inherited cancer syndrome, multigene testing may be more efficient, cost-effective, and have a higher diagnostic yield.^{1,2}

Patients with a Personal and/or Family History Suggestive of Hereditary Cancer May Benefit From CancerNext-*Expanded*®

Genetic testing for hereditary cancer risk should be considered if your patient has a personal or family history of **ANY** of the following*:

CANCER TYPE	MULTIPLE CANCERS OR OTHER CLINICAL RISK FACTORS	EARLY-ONSET CANCERS	ANCESTRY
MALE BREAST	2 OR MORE primary cancers in the	ANY OF THE FOLLOWING	ASHKENAZI JEWISH WITH BREAST
OVARIAN	same person	BEFORE 50 YEARS OF AGE:	CANCER
PANCREATIC	3 OR MORE	Breast, colorectal,	
METASTATIC PROSTATE	cancers on the same side of the family	uterine, gastric	
	10 OR MORE colorectal polyps in a person's lifetime		

CancerNext-Expanded[®] Genes and Associated Cancers

X



77 - 90 gene hereditary cancer test

Medical management guidelines available for most genes



* Ambry Clinician Management Resources (CMRs) are included with test reports and are available on our website.

GENE(S)	ASSOCI	ATED CAI	NCERS**											
	Breast	Ovarian	Uterine	Colorectal & Polyposis	Pancreatic	Prostate	Gastric	Renal/ Urothelial	Endocrine	Central Nervous System	Melanoma	Heme Malignancy	Other	Recessive Disease Associated
AIP										•				
ALK										•				
APC*				•			•		•	•			•	
ATM*	•	•			•									Ataxia telangiectasia
AXIN2*				•										
BAP1*								•			•		•	
BARD1*	•													
BMPR1A*				•			•							
BRCA1*	•	•			•	•								Fanconi anemia
BRCA2*	•	•			•	•					•			Fanconi anemia
BRIP1*		•												Fanconi anemia
CDC73									•				٠	
CDH1 [*]	•						•							
CDK4											•			
CDKN1B*									•	•				
CDKN2A*					•					•	•			
CEBPA [*]												•		
CHEK2*	•					•		•						
CTNNA1							•							
DDX41*												•		
DICER1		•								•				
EGFR													•	
EPCAM [*]		•	•	•	•	•	•	•		•			•	CMMRD ⁺⁺
ETV6*												•		
FH [*]								•	•				•	FH deficiency
FLCN*								•						
GATA2*												•		
GREM1"				•										
HOXB13*						•								
KIT													•	
LZTR1										•				
MAX*									•					
MBD4*				•								•		
MEN1*										•			•	
MET*								•						
MITF											•			
MLH1*		•	•	•	•	•	•	•		•			•	CMMRD ^{††}
MSH2*		•	•	•	•	•	•	•		•			•	CMMRD ^{††}
MSH3*				•†										Polyposis
MSH6*			•	•										CMMRDtt
MIITVH				• t	•					-			•	Polynosis
NF1	•			•					•				•	1 01300315
NF2										•			-	
NTHL1*				• †										Polyposis
														7

gene(s)	ASSOCI	ATED CA	NCERS**											
	Breast	Ovarian	Uterine	Colorectal & Polyposis	Pancreatic	Prostate	Gastric	Renal/ Urothelial	Endocrine	Central Nervous System	Melanoma	Heme Malignancy	Other	Recessive Disease Associated
PALB2*	•	•			•									Fanconi anemia
PDGFRA													•	
PHOX2B										•				
PMS2*		•	•	•	•	•	•	•		•			•	CMMRD ⁺⁺
POLD1 [*]				•										
POLE*				•										POLE-deficiency syndrome
POT1											•	•	•	
PRKAR1A									٠	•			•	
PTCH1										•			•	
PTEN*	•		•	•				•	•	•	•			
RAD51C*	•	•												Fanconi anemia
RAD51D*	•	•												
RB1										•	•		•	
RET*									•					
RPS20*				•										
RUNX1*												•		
SDHA'								•	•				•	Mitochondrial deficiency syndrome
SDHAF2"									•					
SDHB*								•	•				•	Mitochondrial deficiency syndrome
SDHC*								•	٠				•	
SDHD*								•	•				•	Mitochondrial deficiency syndrome
SMAD4*				•			•							
SMARCA4		•								•			•	
SMARCB1										•			•	
SMARCE1										•				
STK11*	•	•	•	•	•		•						•	
SUFU										•			•	
TMEM127*									•					
TP53*	•			•						•		•		
TSC1*								•		•			•	
TSC2*								•		•			•	
VHL*								•	•	•				Polycythemia
WT1								•					•	

Optional CancerNext-*Expanded* Add-ons:

Limited evidence genes (8): ATRIP, EGLN1, KIF1B, MLH3, PALLD, RAD51B, RNF43, TERT There is limited evidence to support a causal role for these genes in association with cancer predisposition. Pancreatitis genes (5): CFTR, CPA1, CTRC, PRSS1, SPINK1

This figure depicts primary cancer associations and may not specify all gene-disease associations. Gene-disease associations and risk estimates vary from study to study, and data are rapidly evolving. Biallelic/Autosomal recessive colorectal cancer risk only **

^{††} CMMRD = constitutional mismatch repair deficiency

Results of Genetic Testing May Inform Personalized Medical Management

The potential benefits of genetic testing for hereditary cancer include*:



+RNAinsight

Paired DNA/RNA genetic testing with +RNAinsight analyzes functional RNA data to help classify DNA variants. It also identifies deep-intronic mutations that may go undetected with a DNA only or reflexive RNA testing approach. As a result, diagnostic yield is higher and variant of uncertain significance rate is lower, providing clarity for patients and healthcare providers.¹This novel functional evidence is especially important in non-White populations that have been underrepresented in research and clinical testing.



For Maximum Flexibility

Order +RNAinsight with analysis of any genes on the hereditary cancer menu*



Scan here to learn more about +RNAinsight.

Let us be your trusted partner

All hereditary cancer tests use Ambry's Classifi® program, a proprietary, knowledge-driven engine for gene classification, variant analysis & interpretation, and reporting. The Classifi program delivers the highest quality test results and ensures we leave no stone unturned in getting answers for you and your patients.



REFERENCES

- Referenced with permission from the NCCN Clinical Practice Guidelines in Oncology (NCCN Guidelines[®]) for Genetic/Familial High-Risk Assessment: Breast, Ovarian, and Pancreatic. V1.2025. ©National Comprehensive Cancer Network, Inc. 2024. All rights reserved. Accessed September 13, 2024. To view the most recent and complete version of the guideline, go online to NCCN.org. NCCN makes no warranties of any kind whatso-ever regarding their content, use or application and disclaims any responsibility for their content.
- application or use in any way. Referenced with permission from the NCCN Clinical Practice Guidelines in Oncology (NCCN Guidelines[®]) for Genetic/Familial High-Risk Assessment: Colorectal, Endometrial, and Gastric. V2.2024 @National Comprehensive Cancer Network, Inc. 2024. All rights 2 reserved. Accessed October 8, 2024. To view the most recent and complet version of the guideline, go online to NCCN.org. NCCN makes no warranties of any kind whatsoever regarding their content, use or application and disclaims any responsibility for their ap-
- Dilcation or use in any way. Garutti, M. et al. 2023. Genes. 2023; 14(5):1025. Genes. 2023; 14(5):1025. https://doi. org/10.3390/genes14051025 Gerkes, E. H.,et al. 2016. Neurogenetics, 17(2), 83–89. https://doi.org/10.1007/s10048-3
- 4. 015-0472-y

- 5.
- Greengard EG, et al. 2010 Jan 5 [Updated 2024 May 23]. In: GeneReviews® [Internet]. Seattle (WA): University of Washington, Seattle; 1993-2024. Available from: https:// www.ncbi.nlm.nih.gov/books/NBK24599/ Korbonits M, et al. 2012 Jun 21 [Updated 2020 Apr 16]. In: GeneReviews® [Internet]. Se-attle (WA): University of Washington, Seattle; 1993-2024. Available from: https://www. ncbi.nlm.nih.gov/books/NBK97965/ Jehenger DD et el. 2000. Int 18 (Updated 2022 Sea 21). Jac GeneReviews® [Internet]. Se-6
- 7
- ncbi.nlm.nih.gov/books/NBK97965/ Lohmann DR, et al. 2000 Jul 18 [Updated 2023 Sep 21]. In: GeneReviews® [Internet]. Se-attle (WA): University of Washington, Seattle; 1993-2024. Available from: https://www. ncbi.nlm.nih.gov/books/NBK1452/ Nemes K, et al. 2017 Dec 7 [Updated 2022 May 12]. In: GeneReviews® [Internet]. Seattle (WA): University of Washington, Seattle; 1993-2024. Available from: https://www.ncbi. nlm.nih.gov/books/NBK469816/ Rafei, H., et al. 2019. Clinical haematology, 32(2), 163–176. https://doi.org/10.1016/j. beha.2019.05.001 Weese-Mayer DE et al. 2004 Jap 28 [Updated 2021 Jap 2021 Ja
- 9.
- bena.2019.05.001 Weese-Mayer DE, et al. 2004 Jan 28 [Updated 2021 Jan 28]. In: GeneReviews® [In-ternet]. Seattle (WA): University of Washington, Seattle; 1993-2024. Available from: https://www.ncbi.nlm.nih.gov/books/NBK1427/ 10

One Enterprise, Aliso Viejo, CA 92656 USA Toll Free +1.866.262.7943 Fax +1.949.900.5501 ambrygen.com ©2025 Ambry Genetics Corporation. All rights reserved. Ambry Genetics® is a registered trademark of Ambry Genetics Corporation. MKT-ONCO-ELYB-20128-EN v3 04 04 25